MANUEL D'UTILISATION / USER'S MANUAL / دليل الاستخدام

ANDINE 3

Pèse-bébé et enfant
Baby to toddler scale
ميزان للرضيع وللطفل

REF EHANDI3

Garder le manuel d'instructions à portée de main en permanence pour pouvoir vous y référer ultérieurement.
Please keep the instruction manual at hand all the time for future reference.
احتفظ بالدليل في متناول يديك بشكل دائم لكي تستطيع الرجوع إليه لاحقاً.

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TABLE OF CONTENTS

GENERAL INFORMATION.....	15
SAFETY INSTRUCTION.....	15
ENVIRONMENTAL.....	15
CLEANING.....	16
MAINTENANCE.....	16
WARRANTY-LIABILITY.....	16
DISPOSING OF THE SCALE.....	16
PREPARE THE SCALE FOR USE.....	17
INSTRUCTION OF USING THE SCALE.....	18
SPECIFICATIONS.....	25
TROUBLE SHOOTING.....	25

GENERAL INFORMATION

Thank you for choosing this product. All features of this product were designed to state of the art and are optimized for simple and straight forward use. If you have any queries or experience any problems not addressed in the operating instructions, please contact your service partner.

We strongly recommend you use the scales on flat and hard surface.

Any soft surface, like carpet will cause inaccuracy.

SAFETY INSTRUCTION

Before putting the device into use, please read with care the information given in the user manual. They contain important instructions for installation, proper use and maintenance of the device.

The manufacturer shall not be liable for damages arising out of failure to heed the following instructions:

- *The scale is for family / home use only and not suitable for professional use. It must be used under adult supervision.*
- *Never leave a child unattended on the scale, so as to prevent the risk of winding around the neck of the height measuring cord of the scale.*
- Do not place the unit on slippery surfaces.
- Batteries should be kept away from small children. If swallowed, promptly seek medical assistance.
- Ensure the tray is properly locked before placing the baby on the scale.
- This product contains small accessories, please keep away from children, so as to prevent inadvertently swallowing the danger!
- When using electrical components under increased safety requirements, always comply with the appropriate regulations.
- Improper installation will render the warranty null and void.
- This device is designed for use indoors only.
- The device meets the requirements for electromagnetic compatibility. Do not exceed the maximum values specified in the applicable standards.

If you have any problem, contact your local service partner.

ENVIRONMENTAL

Battery disposal EC Directive 2006/66/EC.

Batteries are not a part of your regular household waste. You must return batteries to your municipality's public collection or wherever batteries of the respective type are being sold.

Disposal of electric and electronic devices EC Directive 2012/12/EU.

This product is not to be treated as regular household waste but must be returned to a collection point for recycling electric and electronic devices. Further information is available from your municipality, your municipality's waste disposal services, or the retailer where you purchased your product.

CLEANING

We would recommend using alcohol-based wipes or similar when cleaning the scales.

Please do not use large amounts of water when cleaning the scales as this will cause damage to the scales electronics, you should also refrain from using corrosive liquids or high-pressure washers.

MAINTENANCE

The scale does not require any routine maintenance. However, we recommend checking the scale's accuracy at regular intervals. The regularity of these checks is dependent on the level of use and the state of the scale. If any inaccuracies occur, please contact your local dealer or service partner.

WARRANTY-LIABILITY

Thank you for purchasing a Holtex product. This product is guaranteed for 1 year from the date of purchase, provided it is used and cared for correctly, in accordance with the user's manual.

During this guarantee period, and subject to normal use of the product, Holtex will repair or replace the defective product without billing parts or labour.

The manufacturer accepts no liability for material or non-material damage or loss. The guarantee is voided in the event of improper use and the guarantee does not cover:

1. Transport costs and risks.
2. The cost of repairs and/or faults resulting from repairs done by anyone other than Holtex After-Sales staff.
3. The cost resulting from non-acceptance of a complaint (such costs will be billed).
4. If the product's markings such as the model number or date of manufacture have been changed, removed or moved or are illegible, maintenance is not free of charge.

For the addresses, refer to the packaging or product documentation or your specialist retailer.

Repair or replacement under guarantee does not extend or renew the guarantee period.

The guarantee only applies if the complete product is returned along with the bill or original sales receipt issued in the name of the consumer by the retailer, who must have filled in the After-Sales form available for download at www.holtex.fr.


DISPOSING OF THE SCALE

This product is not to be treated as regular household waste but should be handed in to an electrical/electronic equipment recycling centre.


You can obtain further details from your local council, your municipal waste disposal company or the firm which you purchased the product.

PREPARE THE SCALE FOR USE


Remove battery cover on the back of the scale and install 3 x AAA batteries.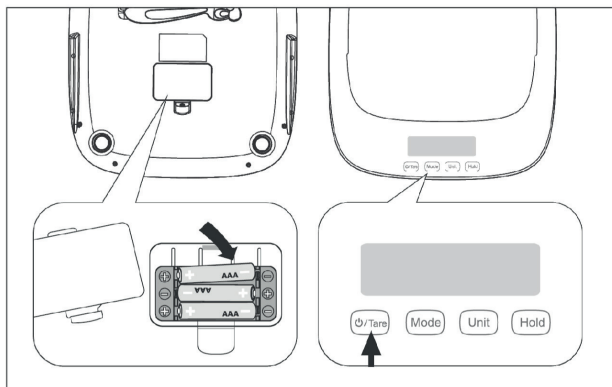

How to attach the tray to the scale?

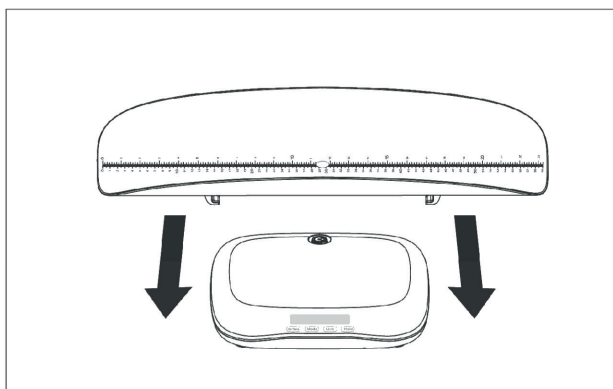Slide the tray until it well fixed to the platform.

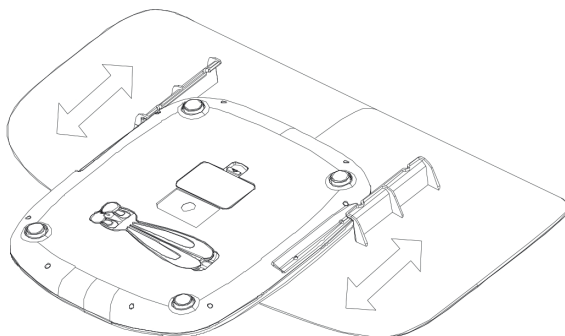INSTRUCTION OF USING THE SCALE

1. Press **(Tare)** button, the display shows **88888** for about 1 s.
2. Wait until the display shows "0:0.0" or "0.00".

Note - If the display shows "0:0.0", the scale is set to pounds/ounces.

If the display shows "0.00", the scale is set to kilograms/grams.

Press **(Unit)** button to choose pounds (0:0.0) or kilograms (0.00).

3. Lay the baby on the tray and read the display. If the baby is moving, press **(Hold)** button to lock in the correct weight.

When weight reading is taken, press **(Hold)** to release weight, the scale will automatically restart to shows "0:0.0" or "0.00" for next weighing after 3s.

4. If there is no weight or any operation, the scale will automatically turn off after 15s.

NORMAL MODE

Normal mode is designed for standard weighing requirement. User can measure weight of baby and tare the weight of rug or diaper placing on the tray. You can use **(Hold)** button to lock in the weight of moving baby and use **(Hold)** button to release locked weight again.

NORMAL MODE DISPLAY

MILK INTAKE MODE

Milk intake function is designed for people who need to know how much milk baby had consumed after feeding milk.

MILK INTAKE MODE DISPLAY


BABY MOMMY MODE

Baby mommy mode is designed for people who can easily deduct their weight and measure baby's weight while cuddling the baby.


BABY MOMMY MODE DISPLAY

SWITCH BETWEEN DIFFERENT MODES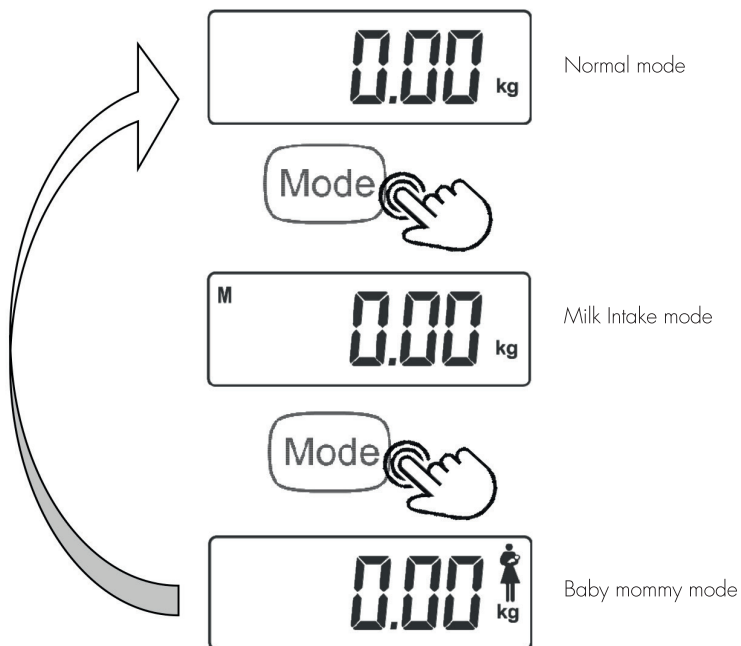

MILK INTAKE MODE

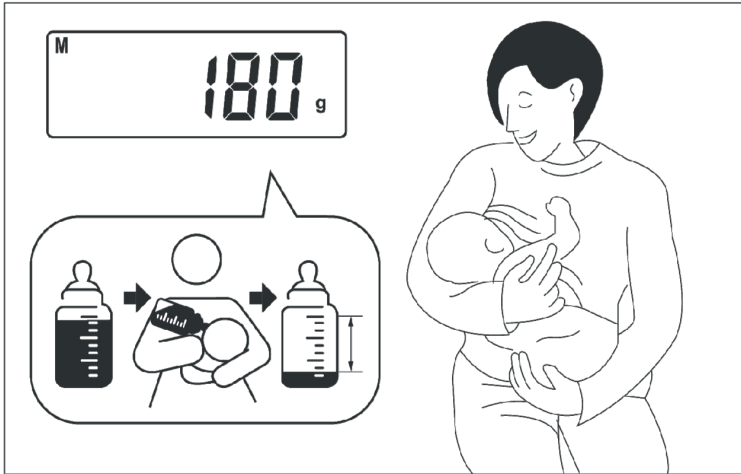Milk Intake function will always store current weight which is placed on the tray and deduct last stored weight and display the differences.


1. Power on the scale and switch to Milk Intake mode with M on the left upper corner.

2. Gently place baby on the tray and press **Hold** button to lock the weight of baby and store baby weight in the scale memory.

When you take baby from tray. It will turn off automatically while without any operation within 15 secs.

3. Then you can feed the baby milk now.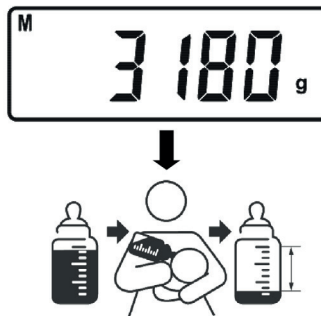

NOTE: When you remove baby from the tray. LCD will display weight difference which deduct from current weight and last stored weight. You can ignore it as you have not feed the baby milk yet.

4. After feeding milk, turn on the scale and switch to Milk Intake mode, then gently place the baby on the tray and press **Hold** button to store the weight.


(example)

5. When weight is stored in the scale memory, cuddle the baby from the tray. Then LCD will display the weight difference (between the current weight and the last weight).


(example)

BABY MOMMY MODE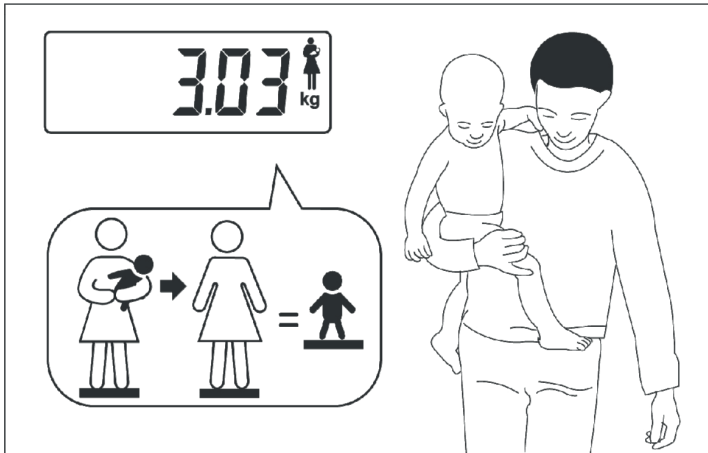


1. Remove the baby tray before turn on the scale.

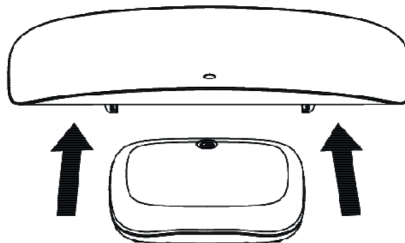2. Press **On/Tare** to turn on the scale and press **Mode** to switch mode to BABY MOMMY mode.


3. Step onto the scale. Stand still until the weight double flash to be locked.

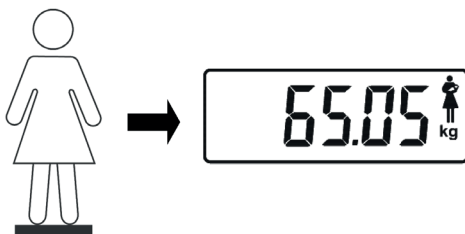4. Step off the scale and the weight restore to zero automatically.

5. Hold the baby in your arms and step onto the scale, wait until the weight double flash to be locked, then step off the scale.

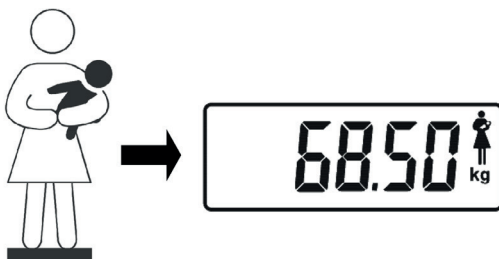
6. After stepping off the scale, display will show the baby weight automatically.


HEIGHT MEASURING MODE

1. Remove baby tray before power on the scale.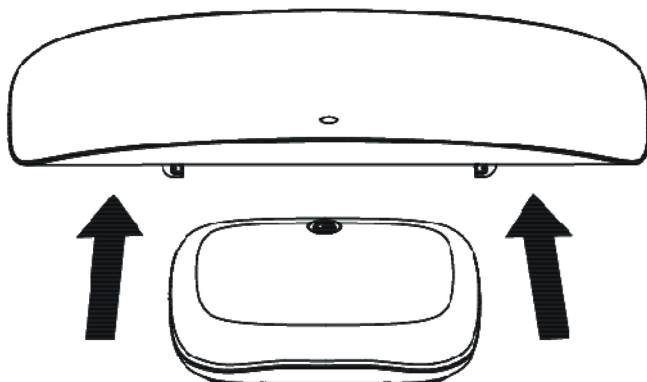

2. Upside down the scale. Then remove head stopper from the bottom of the scale.

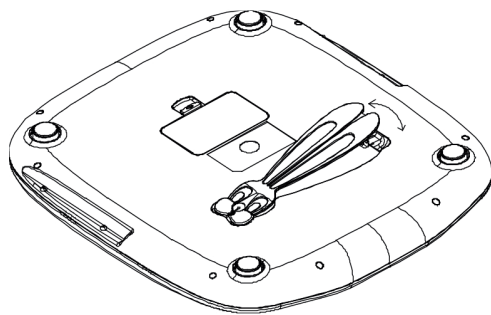
3. Place the scale on a hard and flat surface.


4. Press  to power on the scale.

5. Turn the button anticlockwise and pull height cord out.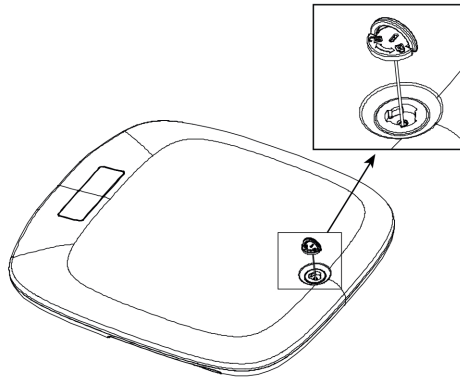


6. Clip the height cord to the head stopper.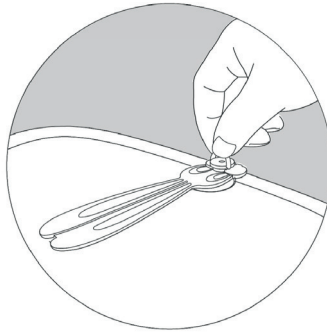

7. Pull height cord up and let the kid step onto the platform. Measure height and take reading from display between 30~ 160 cm, 1 ft ~ 5 ft 3 inches.

Press **Hold** to lock height readings.

Press **Hold** to release height readings again.

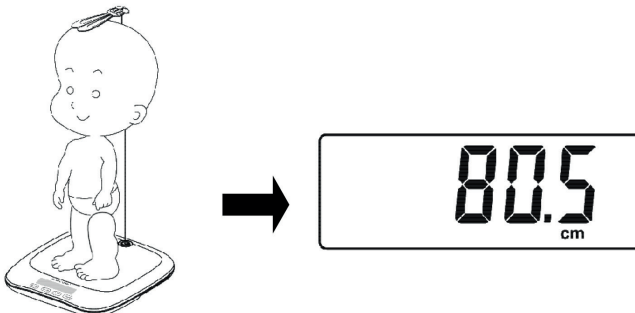SPECIFICATIONS

Model	BCS-9019 / ref. EHADI3	
Capacity	0 ~ 20 kg / 44 lbs, 20 ~ 100 kg / 220 lbs	
Graduation	0.01 kg / 0.2 lb, 0.05 kg / 0.1 lb	
Measuring unit	kg or lb	
Measuring Height	30 ~ 160 cm, 1 ft ~ 5 ft 3 inches	
Weight	1.6 kg	
Dimensions	Platform	290 x 326 x 41 mm
	Tray	600 x 295 x 45 mm
	Scale	600 x 354 x 91.5 mm
Power supply	3 x AAA batteries	
Display type	LCD with blue backlit	

Key functions:

	Power on/off, zeroing the scale, tare weight of subject to get net weight.
	Switch between normal mode, milk intake and baby mommy modes.
	Switch between kg or lb unit. When using height measuring function, unit will be between cm or inch/ft.
	Lock the moving subject like baby or human body.

TROUBLE SHOOTING

Indication	Error Message	Action
	Low battery	Please replace new AAA 1.5v alkaline batteries x 3
	Minus weight	Please remove object on the tray/platform, restart the scale
	Over load	Please remove object on the tray/platform, make sure there is no any object on the tray/platform.
	Over length	Please release height cord.
 	Zero value exceed or lower than default value	Recalibrate the scale.
	EEPROM fail	Please contact your local seller or distributor for further assistance.

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Krell Precision (Yang Zhou) Co., Ltd.